


This kit provides a range of afocal lenses with varying magnifications, allowing you to subjectively equalize image size for your challenging aniseikonia patients.

EVALUATION PROCEDURE

- 1. Initial Setup:** Have your patient wear a current pair of prescription glasses.
- 2. Over-Lens Testing:** Each trial lens has a stamped magnification/minification percentage. For magnification, have the convex side away from the eye. For minification, have the convex side towards the eye. Lenses can be stacked to create magnification percentages larger than 7%.

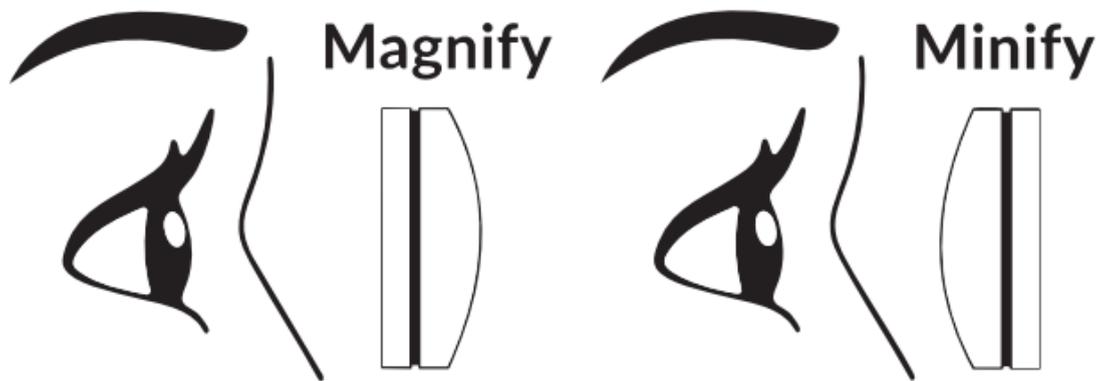

LEARN MORE

Scan the QR code below for our online quick guide and Product Spotlight video:



HOW TO ORDER

When ordering, specify the amount of magnification, and which eye it applies to. This is usually put in the notes of the script. Find our order form at chadwickoptical.com/order-form

	Sphere	Cyl	Axis
OD	-2.00		
OS	-1.5 + 5°		

CHADWICKOPTICAL.COM
phone: 800-410-1618 fax: 800-468-9301
csr@chadwickoptical.com